

# CardioPet\* ECG Device

## Quick Reference Guide



### Record and submit a CardioPet ECG

1. Click **Select & Manage Patient** , and then choose or add the patient.
2. Attach CardioPet\* comfort clips to the patient.
3. Turn on the CardioPet ECG device.
4. Click **Start ECG**  to begin the live ECG stream. This activates wireless communication; the light on the device will illuminate.
5. Click **Record ECG** ; a timer appears in the upper left corner of the window.

Note: Recording stops automatically after 30 or 60 seconds (configurable in **Settings**).

**Important:** Clicking **Record** again will interrupt the recording.

6. Click **Stop ECG**  to stop the live ECG stream.
7. Click any of the following to send the ECG for interpretation:
  - **Upload and Finish Now** .
  - **Upload and Finish Later** .
  - **Send to PACS** , and then from the IDEXX-PACS\* Imaging Software send the ECG to VetMedStat.
8. Complete the case submission on **vetmedstat.com**.

**For additional help, visit [Help/Resources on vetmedstat.com](https://vetmedstat.com) or call 1-800-726-1212.**