Using the Catalyst* Total T4 Test

Storage and handling requirements

- Store in the refrigerator. Do not freeze.
- No warming required—run directly from the refrigerator.
- Total T4 (TT4) slides and reagent can be stored in their pouch at room temperature for up to 8 hours. After 8 hours, store any unused materials in the refrigerator.
- Recommended sample volume:
 - Whole blood: 600–800 µL
 - Serum/plasma: 65 µL (300 µL if running with other slides)

Frequently asked questions

Question	Answer									
Is the Catalyst* Total T ₄ Test just a slide?	No, the Catalyst* Total T ₄ Test contains a single slide and a reagent (conjugate, wash, and substrate). Both components must be run together for each sample run and then discarded.									
What sample types can be run on the total T ₄ test?	Compatible sample types include serum, lithium heparin plasma, and whole blood using the Catalyst* Lithium Heparin Whole Blood Separator.									
What species can be run on the total T ₄ test?	Canine and feline species are supported with total T ₄ .									
Can the total T ₄ test be run with other slides?	Yes! The total T ₄ test can be run alone or with other slides as part of a comprehensive patient profile. For example, you could run a Catalyst* Lyte 4 CLIP, a Catalyst* Chem 17 CLIP, and a total T ₄ test with one patient sample.									
When running with other slides, should the total T ₄ test be loaded in a particular order?	Catalyst One: When running total T ₄ with other chemistry slides, the total T ₄ slide can be loaded in any order. Catalyst Dx: If there are 18 slides or less in the run, the total T ₄ slide can be loaded in any order. For Catalyst Dx runs containing more than 18 slides, load the total T ₄ slide last (on the top of the slide stack). For more information about the slide order for other chemistries, see the <i>Catalyst Chemistry Analyzer Operator's Guide</i> .									
At what range will the total T ₄ test report a numerical result?	<table><tr><th>Species</th><th>U.S. Units</th><th>S.I./French S.I. Units</th></tr><tr><td>Canine</td><td>0.5–10.0 µg/dL</td><td>6.4–128.7 nmol/L</td></tr><tr><td>Feline</td><td>0.5–20.0 µg/dL</td><td>6.4–257.4 nmol/L</td></tr></table>	Species	U.S. Units	S.I./French S.I. Units	Canine	0.5–10.0 µg/dL	6.4–128.7 nmol/L	Feline	0.5–20.0 µg/dL	6.4–257.4 nmol/L
Species	U.S. Units	S.I./French S.I. Units								
Canine	0.5–10.0 µg/dL	6.4–128.7 nmol/L								
Feline	0.5–20.0 µg/dL	6.4–257.4 nmol/L								
What is the run time for the total T ₄ test?	Total T ₄ test results are available in approximately 15 minutes after the start of the run.									
Is it okay to dilute samples that are undergoing total T ₄ testing?	IDEXX does not support the use of diluted samples when running the Catalyst Total T ₄ Test.									
How often can total T ₄ tests be left at room temperature and then returned to the refrigerator?	Once at room temperature, total T ₄ tests can be returned to the refrigerator up to 5 times as long as the foil pouch is unopened.									
What if a total T ₄ test is accidentally frozen?	Any tests suspected of being frozen should be allowed to thaw at room temperature for at least 30 minutes before use.									
Should I run quality control for total T ₄ ?	Quality control is not required but is available through IDEXX Technical Support.									

For more information about running the Catalyst Total T4 Test, consult the operator's guide for your Catalyst Dx* Chemistry Analyzer or Catalyst One* Chemistry Analyzer. For additional information, please contact IDEXX Technical Support.

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© 2015 IDEXX Laboratories, Inc. All rights reserved. • 06-0005042-02
*Catalyst, Catalyst Dx, and Catalyst One are trademarks or registered trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.