

No specimen? No problem.

Use IDEXX completion test codes to submit your specimen at a later date.

It happens. You place an order for an IDEXX Reference Laboratories profile, and—for whatever reason—you are unable to obtain the required specimen to submit. With IDEXX completion test codes, you can submit specimens for certain IDEXX Reference Laboratories profiles for up to four weeks (28 days) following the original submission date.

How to use IDEXX completion test codes:

Completion test codes are an effortless way for you and your team to ensure that your IDEXX Reference Laboratories order is completed.

1. Confirm that it has been less than 28 days since the original requisition submission and that the patient's first and last names match the exact spelling that appeared on the original requisition to avoid unexpected charges.
2. Begin a new requisition. Locate the completion test code below (or in the online test directory in VetConnect® PLUS) for the specimen you are submitting.
3. Submit the newly created test requisition form with the specimen and the corresponding completion test code. Submitting a copy of the original requisition form may result in a testing delay.

Note: For completion of Cancer Dx as part of a profile, both a EDTA whole blood specimen and serum specimen is required and must be received together with a single requisition form with the appropriate Cancer Dx completion test code ordered.

If you use the IDEXX FecalChek® Home Collection Kit to submit a fecal completion test code, then you must submit a new requisition form. The new requisition form should contain only the fecal completion test code and should accompany the specimen submitted by your client. Please do not submit fecal completion codes using a copy of the original requisition form on which other laboratory testing was previously ordered as doing so may result in a testing delay.

Example: Your original IDEXX Reference Laboratories test order contained a urinalysis component, but a urine specimen was not provided with the original submission. On a new requisition, you would simply use completion test code UAAC (Urine to Complete Urinalysis) to complete your order. This easy-to-use process can be utilized for up to four weeks following the original submission date. Beyond four weeks (28 days), use the billable test code UNA.

IDEXX COMPLETION TEST CODES		
Code	Name	Specimen requirements
CANDXAC	Serum and Whole Blood to Complete IDEXX Cancer Dx™ Test—Canine	1 mL LTT and 2 mL serum
CYSBAC*	Urine to Complete IDEXX Cystatin B Test—Canine/Feline	5 mL urine in a sterile container
FDXAC	Fecal to Complete Fecal Dx™ Antigen Panel	3–5 g fresh feces in a clean plastic container
FDXGAC	Fecal to Complete Fecal Dx™ Antigen Panel with <i>Giardia</i>	3–5 g fresh feces in a clean plastic container
FDXGPAC	Fecal to Complete Fecal Dx™ Antigen Profile with <i>Giardia</i>	3–5 g fresh feces in a clean plastic container
FDXPAC	Fecal to Complete Fecal Dx™ Antigen Profile	3–5 g fresh feces in a clean plastic container
FLTAC	Fecal to Complete Fecal Ova & Parasites	3–5 g fresh feces in a clean plastic container
FLTGAC	Fecal to Complete Fecal Ova & Parasites with <i>Giardia</i>	3–5 g fresh feces in a clean plastic container
T4AC	Serum to Complete T ₄ —Canine/Feline/Equine	1 mL serum
UAAC	Urine to Complete Urinalysis	5 mL urine in a sterile container
UACBAC	Urine to Complete Urinalysis with IDEXX Cystatin B Test—Canine/Feline	5 mL urine in a sterile container
UAPCAC	Urine to Complete Urinalysis with IDEXX Cystatin B Test and UPC—Canine/Feline	5 mL urine in a sterile container
UAUPCAC	Urine to Complete Urinalysis with Urine Protein/Creatinine Ratio (UPC)	5 mL urine in a sterile container
UAPCRFAC	Urine to Complete Urinalysis with Reflex UPC (if Indicated)—Canine/Feline	5 mL urine in a sterile container
UNAPCRFAC	Urine to Complete Urinalysis with IDEXX Cystatin B Test and Reflex UPC (if Indicated)—Canine/Feline	5 mL urine in a sterile container
UPCAC	Urine to Complete UPC	5 mL urine in a sterile container
UPCCBAC	Urine to Complete UPC with IDEXX Cystatin B Test—Canine/Feline	5 mL urine in a sterile container

Please check that these codes are linked properly in your practice management software. Keep this information handy and share with your entire team.

*Code used to complete the IDEXX Cystatin B Test portion of IDEXX Anywhere (S-code) profiles