

Welcome to the latest improvements in VetConnect* PLUS

Coming soon—a new look for VetConnect PLUS test results

Based on customer feedback, we've adjusted the overall design and colour palette for test results on vetconnectplus.ca. Reviewing diagnostic results will be quicker, more efficient, and easier on the eye.

The screenshot shows the VetConnect PLUS test results interface for a patient named REESE, 13745, a 15-year-old female domestic short-haired cat. The interface includes a header with patient information, a navigation bar with tabs for 2017, 2016, and 2015, and a main content area displaying chemistry test results. Annotations highlight key features:

- Patient signalment in header:** Points to the patient name, ID, and species.
- Graphing control:** Points to the 'Graphing' toggle switch.
- Individual trending chart:** Points to the 'BUN' test result, which has a small chart icon next to it.
- Consolidated result details:** Points to the 'Result Details' dropdown menu.
- Command buttons:** Points to the 'Print', 'Share', 'Consult', and 'Summary' buttons.

The chemistry test results table shows the following data:

Test	12/8/17 2:59 PM	8/28/17 3:02 PM	3/2/17 3:35 AM	10/13/16 4:50 AM
Glucose	176	123	117	151
IDEXX SDMA	18	20	15	19
Creatinine	2.3	2.6	2.4	2.1
BUN	42	41	40	39
BUN:Creatinine Ratio	18	16	17.1	18.6
Phosphorus	4.4	5.1	4.2	4.4
Calcium	9.9	10.5	10.1	10.4
Sodium	159	159	151	151
Potassium	4.0	4.1	4.6	4.2
Na:K Ratio	40	39	33	36
Chloride	121	118	119	117
Total Protein	7.9	8.4	7.4	7.9

The 'BUN' test result is highlighted with a red circle, indicating its individual trending chart. Below the table, a description and common causes for decreased total protein are provided.

One-click access to differential diagnosis, testing algorithms, and other workup tools


Command buttons have been readjusted so you can easily:

- Print test results.
- Share results through email or Petly* online pet pages.
- Request a complimentary consult from one of our board-certified internal medicine specialists.
- Access a client-friendly summary to share with a client.
- Mark results as read or unread.

Review other recent improvements below

Save the tests that matter most to you

You can save tests as you order them in the IDEXX Diagnostics window or in the Directory of Services.

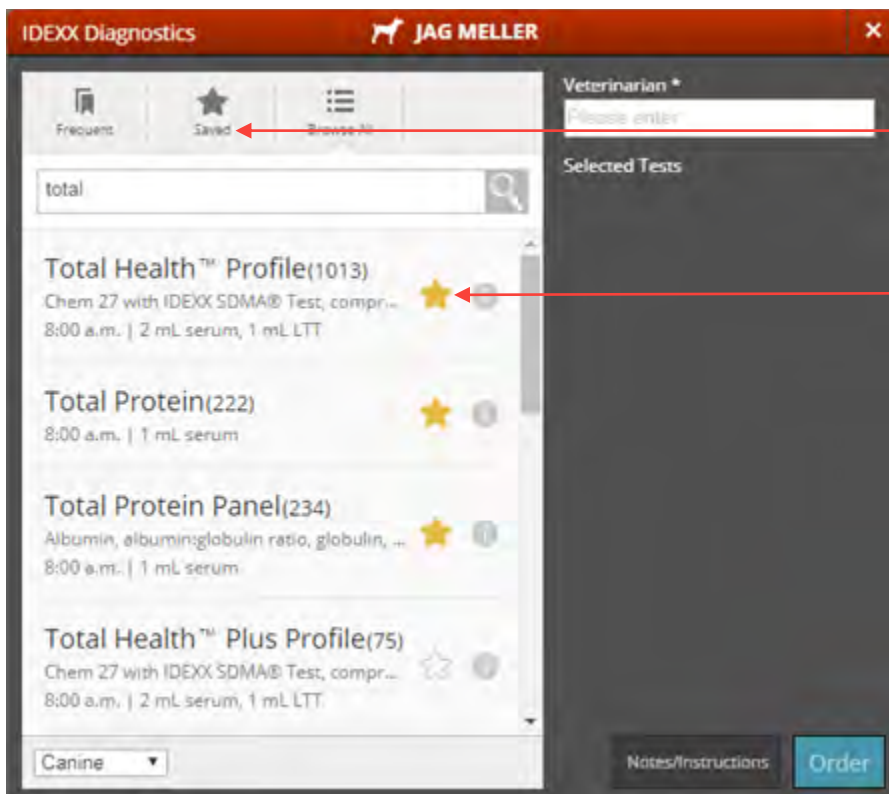


To find and retrieve a saved test, click Saved.

Click the star ★ to save a test. You can save as many tests as you'd like.

Print or email results from the Home page or the search results page

Print or email a single result

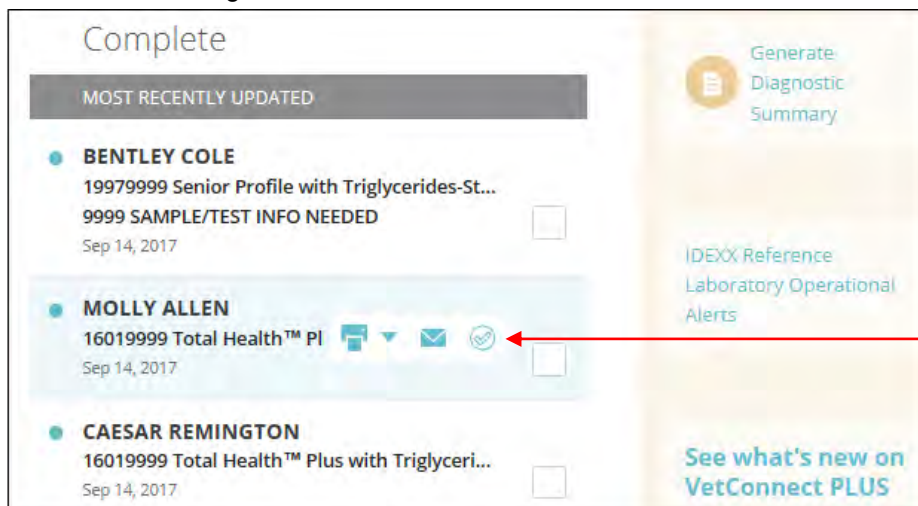

Hover the cursor over a test result to display print, email, and mark-as-read buttons.

Print or email multiple results

<input checked="" type="checkbox"/>	KANEL HONTORIA SNAPshot Dx Analyzer	582-1	OSCAR PAEZ CASTRO	Req: 118	Apr 10, 2013 12:29 pm	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	DUTCH PEREZ SNAPshot Dx Analyzer	598-1	OSCAR PAEZ CASTRO	Req: 141	May 2, 2013 7:00 pm	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	MATHEW DIAZ LaserCyte Hematology Analyzer	634-1	Shih Tzu	Req: 198	Jul 13, 2013 10:20 am	<input type="checkbox"/>
<input checked="" type="checkbox"/>	TOKY MONTES Catalyst Dx Chemistry Analyzer	84-1	Shih Tzu	Req: 201	Jul 30, 2013 4:20 pm	<input type="checkbox"/>
<input checked="" type="checkbox"/>	POLO 18339 TERRY 15128 Catalyst Dx Chemistry Analyzer	Yorkshire Terrier	RUBY		Nov 23, 2012 6:55 pm	<input type="checkbox"/>
<input checked="" type="checkbox"/>	1 ProCyte Dx Hematology Analyzer	Maltese			Sep 3, 2012 1:14 pm	<input type="checkbox"/>

Print Email Mark Read Deselect All

Select up to ten results, and then print, email, or mark as read with one click.

Filter results by diagnostic testing type

IDEXX VetConnect⁺ PLUS

Home | Directory of Services | Imaging | Telemedicine

DIAGNOSTICS

9/7/2017 - 9/14/2017

Order

☒ Reference Laboratory ☐ In-house Laboratory ☒ Imaging

All Veterinarians

In Process

MOST RECENTLY UPDATED

☒ JEFFERSON GRESENS
19979999 Senior Profile with Triglycerides-St...
Sep 14, 2017
Partial

Complete

MOST RECENTLY UPDATED

☒ BENTLEY COLE
19979999 Senior Profile with Triglycerides-St...
9999 SAMPLE/TEST INFO NEEDED
Sep 14, 2017

Select an option to filter results by test type.

Track test progress at a glance

In Process

MOST RECENTLY UPDATED

LILLY COOLIDGE 5530-1

4033 Urine Culture and MIC Susceptibility*

Nov 26, 2015

Ordered

MOLLY BOTT 14038-1

VetLyte Electrolyte Analyzer

ProCyte Dx Hematology Analyzer

Catalyst Dx Chemistry Analyzer

Nov 26, 2015

Running

DIVA CHESTER 34718-1

401 Aerobic & Anaerobic Cultures*

Nov 26, 2015

At the Lab

CHARM FOSTER 21277-1

4033 Urine Culture and MIC Susceptibility*

Nov 26, 2015

Partial

✓ Hematology

✓ Chemistry

✓ Urinalysis (updated)

Endocrinology (updated)

New status icons keep you up to date on test status.

Icon...	Applies to...	Indicates....
<div>Ordered</div>	IDEXX Reference Laboratories	Order has been created
<div>At the Lab</div>	IDEXX Reference Laboratories	Order received but no results available yet
<div>Partial</div>	IDEXX Reference Laboratories	Some tests complete; partial results available
<div>Running</div>	IDEXX in-house analyzers	Test is running now; no results available yet

To view partial test results:

Place the mouse pointer over

Partial

 to see which test categories are complete and which are pending. Click any category, partial or complete, to see available test results.

CHARM FOSTER 21277-1

4033 Urine Culture and MIC Susceptibility*

Nov 26, 2015

Partial

✓ Hematology

✓ Chemistry

✓ Urinalysis (updated)

Endocrinology (updated)

In this example, three categories are complete and Endocrinology is pending.

Easily find and add tests to samples already at IDEXX Reference Laboratories

Initial results often indicate more testing is needed. We've made it quick and easy to search all IDEXX tests and add those you need.

1. Open the patient's test results in VetConnect PLUS.
2. Click the new Add to Order button at the top.
The button is active only if add-on tests are available for this order.
3. Search all IDEXX tests, and then select the ones to add.
You'll receive a confirmation message.
4. (Optional) Print the updated test requisition form
for your records.

You don't need to send the printed form to IDEXX. Your test request is sent electronically, and we already have your samples.

The screenshot shows the IDEXX VetConnect PLUS interface for a patient named MACY GILKS. At the top, there is a navigation bar with the year 2017, the date APR 22, and a blue button labeled '+ Add to Order'. Below this, there is a table of test results. The table has three columns: Test Name, Result, and Reference Range. The tests listed are RBC, Hematocrit, and Hemoglobin. The results are 7.6, 0.46, and 177 respectively. The reference ranges are 5.4 - 8.7 x10^12/L, 0.38 - 0.57 L/L, and 134 - 207 g/L. A red box highlights the '+ Add to Order' button.

Test Name	Result	Reference Range
RBC	7.6	5.4 - 8.7 x10 ¹² /L
Hematocrit	0.46	0.38 - 0.57 L/L
Hemoglobin	177	134 - 207 g/L

The screenshot shows a confirmation message on the IDEXX VetConnect PLUS interface. At the top, there is a yellow icon of a beaker with a checkmark. Below the icon, the text reads: "You have requested additional testing to reference laboratory samples". Below this, there is a paragraph of text: "Your request will be processed by IDEXX Customer Support. If you have any questions or need assistance please call, 1-800-667-3411." Below this, there is another paragraph of text: "If you have already submitted samples, you do not need to send a new copy of your requisition form." At the bottom, there are two buttons: "Done" and "Print Requisition".

You have requested additional testing to reference laboratory samples

Your request will be processed by IDEXX Customer Support. If you have any questions or need assistance please call, 1-800-667-3411.

If you have already submitted samples, you do not need to send a new copy of your requisition form.

Done Print Requisition