SNAP* T₄ Test Kit

For use only with the IDEXX SNAP* Reader Analyzer.

Overview

The SNAP T₄ Test Kit is an enzyme-linked immunosorbent assay (ELISA) for the quantitative measurement of total T₄ (thyroxine) in canine, feline (domestic) and equine serum or plasma.¹ For in vitro use only.

SNAP T₄ Test Ranges

To optimize the SNAP T₄ assay's performance, the test has been split into overlapping high and low dynamic ranges.

- T₄ Low Dynamic Range: 0.5–3.5 µg/dL (6.4–45.0 nmol/L)
- T₄ High Dynamic Range: 2.0–7.0 µg/dL (25.7–90.0 nmol/L)

Precautions and Warnings

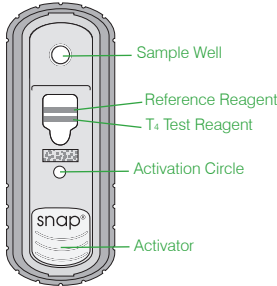
- Run the SNAP calibration device at the beginning of each day.
- This test cannot be visually interpreted. The SNAP Reader analyzer must read all tests.
- Use a separate pipette tip and sample tube for each test.
- Do not use components after their expiration dates.
- Do not mix components from different lots of test kits.
- Do not use a device that does not have two blue stripes visible before adding the sample.
- Keep the device in its sealed foil pack at room temperature (18°–27°C/64°–80°F) for at least 10 minutes before use.
- Prewarming of the conjugate solution is not required.
- Keep the SNAP device horizontal during the entire test procedure.**
- IMPORTANT: Do not depress the activator until instructed.**
- Insert the device into the analyzer immediately after activation.

Storage

- SNAP devices and test reagents must be stored at 2°–7°C (36°–45°F).
- Do not heat or microwave the kit components.
- Take out only the kit components you will need.
- Do not leave kit components at room temperature for more than 8 consecutive hours.

Kit Components

- Conjugate solution
- SNAP test devices
- Disposable pipette tips for the 300-µL manual pipettor
- Disposable sample tubes
- Workstation

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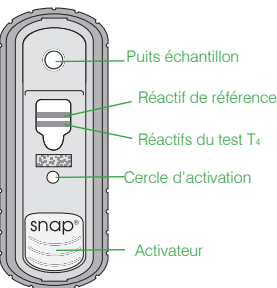
Composants Requirés (Not Provided in Kit)

- VetTest* pipette tips
- 300-µL manual pipettor (provided with the SNAP Reader analyzer)

Sample Information

- Do not use whole blood. Use serum or plasma,¹ either fresh, previously frozen or stored at 2°–7°C (36°–45°F) for up to 7 days. To store longer than 7 days, freeze the sample (-20°C/-4°F or below). Remove serum or plasma¹ from the red blood cells before freezing. Bring previously refrigerated or frozen samples to room temperature (18°–27°C/64°–80°F) before running assays.
- SNAP T₄ sample size requirements: 100 µL for Low Dynamic Range, 50 µL for High Dynamic Range.
- Centrifuge previously frozen or older samples before use.
- Results of this test are not affected by hemolysis or lipemia.

Test Procedure

The VetTest analyzer provides on-screen instructions throughout the analysis. For complete instructions, refer to the Running Quantitative SNAP Tests chapter in the SNAP Reader Operator's Manual.

Test	Sample Volume	Conjugate Volume
Canine Screen	100 µL	300 µL
Canine Monitor	50 µL	300 µL
Feline Screen	50 µL	300 µL
Feline Monitor	100 µL	300 µL
Equine Screen	100 µL	300 µL
Equine Monitor	50 µL	300 µL

Interpreting Test Results

The recommendations below are intended to provide general guidance only. As with any diagnosis, you should use clinical discretion with each patient based on a complete evaluation of the patient, including history, physical examination and complete laboratory profile.

Reference Ranges (µg/dL)			Reference Ranges (nmol/L)				
	Feline	Canine	Equine		Feline	Canine	Equine
Low	<1,0	<0,8	<0,9	Low	<13	<10	<12
Borderline Low		0,8–1,5		Borderline Low		10–20	
Normal	1,0–5,0	1,6–5,0	0,9–2,8	Normal	13–64	21–64	12–36
Borderline High	2,5–5,0			Borderline High	32–64		
High	>5,0	>5,0	>2,8	High	>64	>64	>36
Therapeutic Range		3,0–6,0		Therapeutic Range		39–77	

Note: 1 µg/dL is equal to 12.87 nmol/L.

IDEXX Customer Support

U.S./Canada/Latin America: 1-800-248-2483
Europe: 00800 1234 3399
Australia: 1300 44 33 99
New Zealand: 0800-102-084
Asia: 0800-291-018

If possible, please have your kit lot number available.

¹Lithium heparin-based plasma.

^{*}SNAP and VetTest are trademarks or registered trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.

Covered by U.S. Patent Nos. 5,726,010; 5,726,013; 5,750,333 and/or 6,007,999. Other U.S. and/or foreign patents issued or pending.

© 2012 IDEXX Laboratories, Inc. All rights reserved • 06-17600-01

EC|REP IDEXX Laboratories, Inc. One IDEXX Drive, Westbrook, Maine 04092 USA
IDEXX Europe B.V. PO. Box 1334, NL-2130 EK Hoofddorp

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© 2012 IDEXX Laboratories, Inc. Tutti i diritti sono riservati.